DE

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Seite 2

Funk Infrarot-Bewegungssender

EN

Operating Instructions

Page 8

RF infrared motion transmitter

FR

Mode d'emploi

Page 14

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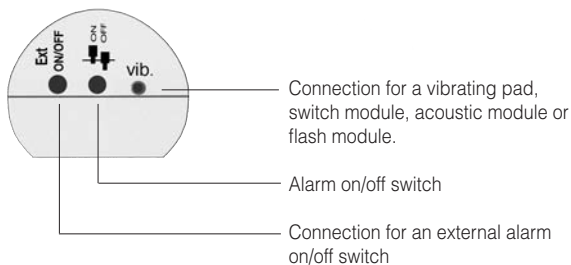
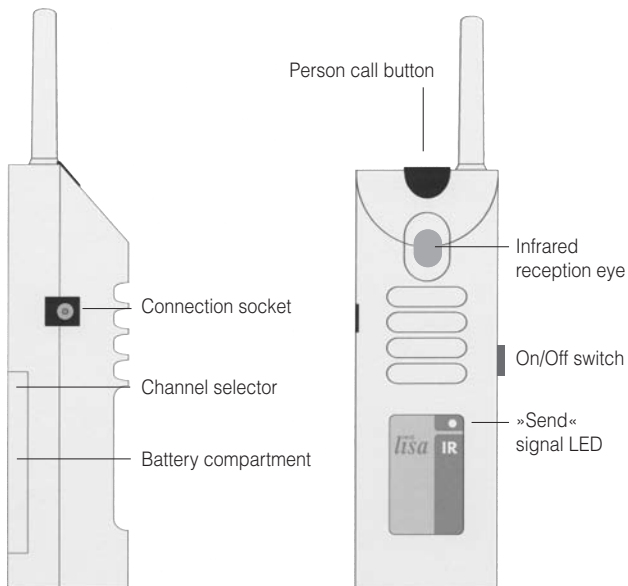
Congratulations on purchasing your *lisa RF Infrared motion transmitter*. We hope that it gives you many years of reassuring operation. Please read through these operating instructions carefully. They tell you how to set up the system correctly and explain all the system's features to you.

Standard components

- *lisa Infrared motion transmitter*
- 9 V battery
- 12 V Plug-in power supply unit
- Operating instructions
- Warranty card

If any parts are missing, please immediately contact your dealer or the manufacturer directly.

Components not included

External alarm on/off switch

Operating principle

A signalling system comprises at least one transmitter and one receiver. The *Infrared motion transmitter* detects the movements of a person or animal within the sensing area and converts them into radio pulses. These radio pulses (868.35 MHz) are transmitted to the receiver without the need for wires.

Special features

When a signal is detected, it is transmitted for a total of 22 seconds. This increases both the reliability and the range of the system.

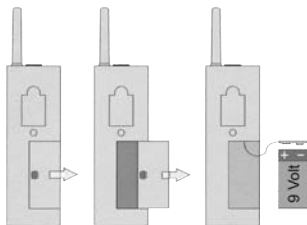
Setting up the system

1. Mains operation

To connect the power supply, plug the connector on the end of the power cord into the corresponding jack on the left-hand side of the device. Then plug the other end of the cord into a normal 230 V wall outlet. It is possible that the transmit signal may be triggered once when you connect the power supply.

2. Inserting the battery

There is a battery compartment on the back of the *Infrared motion transmitter*. Open it by sliding the cover of the battery compartment to the right of the transmitter using your thumb. Connect a 9V block battery to the connector plug, insert the battery into the battery compartment and close the device.

3. Setting the correct radio channel


A special digital transmission process ensures that the radio signals are picked up without errors. This process also makes it possible to set 10 different radio channels.

When supplied from the factory, all *Infrared motion transmitters* are set to channel 0. It is only necessary to set units to other channels if you have several different users operating units in the same building.


Set the channel you want by adjusting the channel selector switch on the left-hand side of the unit to the required channel number. Use a small screwdriver to adjust this switch.

All transmitters and receivers must be set to the same channel.

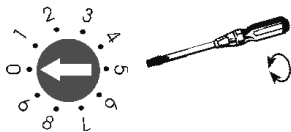Fig. setting the correct radio channel

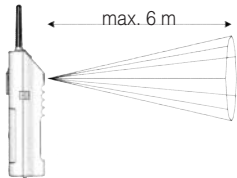4. Installation of the *Infrared motion transmitter*

The *Infrared motion transmitter* can detect movement at a maximum distance of up to 6 meters away.

The opening angle measures approx. 35° to 40°.

Please note the following when setting up the infrared motion transmitter:

1. The infrared reception sensor must be free and clear.
2. The maximum infrared recognition range must be observed.

Switching on the *Infrared motion transmitter*

Use the slider switch on the right-hand side of the device to switch the transmitter on.

There are two ways to activate the infrared reception sensor:

1. Switch on the Alarm ON/OFF switch on the bottom of the transmitter (switch pressed in = IR motion transmitter is activated).
2. Use an external ON/OFF switch.

Once the transmitter has been activated, the infrared reception sensor takes about 30 seconds to switch on to give you time to leave the alarm area. If a person or animal comes within the recognition range of the *Infrared motion sensor*, an alarm signal will sound.

Available accessories

External alarm ON/OFF switch (A-2937-0)

For connection to the *Infrared motion transmitter*. This external alarm ON/OFF switch can be used to enable and disable the transmitter without triggering the alarm (if it is positioned correctly). Please note that the alarm ON/OFF switch on the transmitter must be enabled in order to connect the external alarm ON/OFF switch. Switch for surface-mounting. Length of the connection cable: 10 m.

Call button

Pressing the call button on the *Infrared motion transmitter* causes a signal to be triggered on every receiver which is set to the same channel as the transmitter. This enables people to be called who are in other rooms. Also, this button can be used for checking that the system is functioning properly, even without the accessories connected.

Radio range

Note the following points when using the *Infrared motion transmitter*.

The radio signals are designed to have a range of 80 meters under optimum conditions.

The following conditions may result in a reduced range:

- Signal transmission through building structures or vegetation.
- Interference from TVs, computer monitors and mobile phones. Under certain circumstances, this may prevent the receiver from functioning at all.
- Positioning of the transmitter at an unfavourable distance from surfaces which reflect radio waves, for example floors and walls. This may lead to the signal wave being weakened or even cancelled out by the reflected wave.
- Metallic objects reduce the range due to the shielding effect of the metal.
- Particularly in towns and cities, there may be many other sources of radio waves which can disrupt the original signal.
- Having units operating at similar frequencies located close to one another may also result in mutual interference.

Positioning the radio transmitter

Bear in mind the following points when positioning the radio transmitter, in order to achieve the greatest possible range from the radio transmitter:

- The radio transmitter should not be set up close to devices which emit powerful interference signals, such as TVs, computer monitors or mobile phones.
- The radio transmitter should not be placed on metallic objects, since otherwise the radio range will be restricted by metallic shielding.
- Position the radio transmitter as close to the centre of the building as possible, at a medium height in the room.

Maintenance and care

The *Infrared motion transmitter* does not require any maintenance.

If the unit does become dirty, simply wipe it clean with a soft, damp cloth. Never use spirits, thinners or other organic solvents. Do not set up the *Infrared motion transmitter* where it will be exposed to full sunlight for long periods. In addition, it must be protected against excessive heat, moisture and severe mechanical shocks.

Note: This product is **not** protected against splash water. Do not place any containers filled with water, such as flower vases, or anything with an open flame, such as a lit candle, on or near the product.

Warranty

The *Infrared motion transmitter* is a very reliable product. Should a malfunction occur despite the unit having been set up and operated correctly, please contact your dealer or the manufacturer directly.

This warranty covers the repair of the product and returning it to you free of charge.

It is recommended that you send in the product in its original packaging, so keep the packaging for the duration of the warranty period.

The warranty does not apply to damage caused by incorrect handling or attempts to repair the unit by people not authorised to do so (destruction of the seal on the unit).

Repairs will only be carried out under warranty if the completed warranty card is returned accompanied by a copy of the dealer's invoice/till receipt.

Always specify the product number in any event.

Disposal of used electric and electronic units (applicable in the countries of the European Union and other European countries with a separate collection system). The symbol on the product or the packaging indicates that this product is not to be handled as ordinary household waste but has to be returned to a collecting point for the recycling of electric and electronic

units. You protect the environment and health of your fellow men by the correct disposal of this products. Environment and health are endangered by a faulty disposal. Material recycling helps to reduce the consumption of raw material. You will receive further information on the recycling of this product from your local community, your communal disposal company or your local dealer.

Technical data

| | |
|------------------------------------|----------------------------------|
| Power supply: | via plug-in power supply DC 12 V |
| Power failure operation: | 9 V block battery |
| Frequency: | 868.35 MHz |
| Max. transmission power: | <10 mW |
| Working range: | 0° C – 40° C |
| Height (including the antenna): | 147 mm (201 mm) |
| Width: | 50 mm |
| Depth: | 41 mm |
| Weight (including battery): | 145 g |

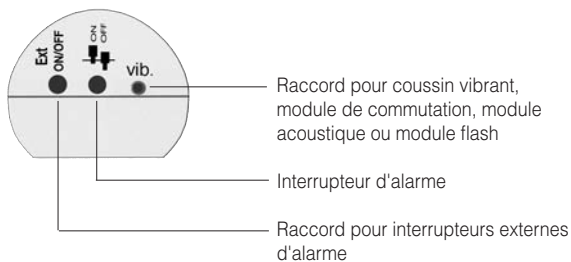
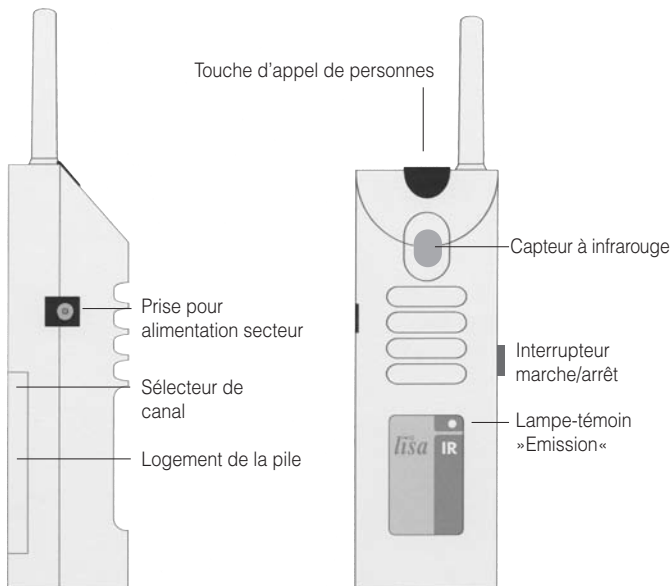
This device satisfies the following EC directives:

- 2011 / 65 / EC RoHS directive
- 2012 / 19 / EC WEEE directive
- 2014 / 53 / EC RED directive

Compliance with the directives listed above is confirmed by the CE seal on the device. CE compliance declarations are available on the Internet at www.humantechnik.com.

Technical specifications subject to change without prior notice.

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