HUMANTECHNIK

DE

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Vibration sensor for mobile phones

This sensor is designed to react to the notifying vibration of mobile phones and tablets, in order to inform you of incoming messages and phone calls via the Humantechnik alerting systems.

Setup

Insert the connector plug of the sensor into socket no. 1 of the Signolux „Universal Acoustic Transmitter“ (A-2656-0) or the socket of the lisaRF „Acoustic Telephone Transmitter“ (A-2464-0).

Regarding Signolux: Program your signolux receivers to indicate a telephone event icon upon signalisation. If necessary, please refer to the user manuals of the transmitter or the receivers.

Please make sure that your device is set to vibrate on incoming calls and messages. Activating the ringtone additionally is beneficial to the detection.

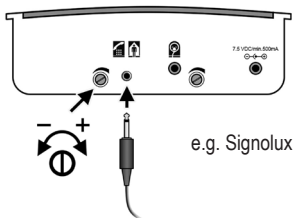
Lay your mobile phone or tablet flat onto any surface and place the sensor directly on top of the device.

Adjustment

Adjust the sensitivity of socket #1 of the detector to a medium setting. If possible, have someone call you or send you a message on the according device for testing purposes.

English

Should the sensor not react reliably, increase the sensitivity. Should false alerts occur, lower the sensitivity accordingly.

Disposal of used electric and electronic units

The symbol on the product or the packaging indicates that this product should not to be handled as ordinary household waste but should be returned to a collecting point for the recycling of electric and electronic units.

You protect the environment and health of your fellow men by the correct disposal of this products.

Environment and health are endangered by a faulty disposal.

Material recycling helps to reduce the consumption of raw material.

You will receive further information on the recycling of this product from your local community, your communal disposal company or your local dealer.

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Humantechnik GHL AG
Sarabec Ltd.